

**REFLEXX S.p.A.Unipersonale** 

INFORMATIVE NOTE

rev 04 del 13 novembre 2019

Product: reflexx 92 art. R92/ S - art. R92/ M - art. R92/ L - art. R92/ XL

Emessa da G.Isetti - Amm.re Unico

Pag.: 1/2

# TRADEMARK REFLEXX 92

### 1. NAME AND ADDRESS OF MANUFACTURER

REFLEXX S.p.A.

Via Passeri, 2

46019 VIADANA (MN)

Tel. 0375 758891 Fax. 0375 464504

#### 2. GENERAL DESCRIPTION OF PRODUCT

- Gloves Latex, five fingers, color natural / amber beaded drip and antiskid surface
- Gloves chlorinated internally for easy donning and also externally
- Gloves joined the pair (right hand and left hand) in a sealed envelope
- multipurpose glove to protect the hand and / or the product handled. It can be used for general purposes
- Unit Sales per pair in envelope
- Outer packagings containing n ° 120 envelopes





Imballo - Pack					
	1	120			
Taglia Size	Articolo Articie	Taglia Size	Cartone Carton		
R92 S	8032891631560	s	8032891636565		
R92 M	8032891631577	м	8032891636572		
R92 L	8032891631584	L	8032891636589		
R92 XL	8032891631591	XL	8032891636596		

# 3. QUALITY AND COMPLIANCE WITH REGULATIONS

As a PPE category I risk according to Reg EU 2016/425. The product is certified according to EN420 e EN ISO 374.

## 4. DIMENSIONS AND PERFORMANCE

Dimensions (in mm.)	Size		Longth (min )	Width in	weight in g	Thickness
	size	size CE	Length (min.) (± 5)	mm. (± 5)	(±3)	(± 0.04)
	S	7	300	90	45	
	M	8	300	100	50	0.20
	L	9	300	110	55	0.30
	XL	10	300	115	60	

EN 420:2003 + A1 2009		in compliance
	compliant	
	measures	in compliance



REFLEXX S.p.A.Unipersonale INFORMATIVE NOTE		rev 04 del 13 novembre 2019			
Product: reflexx 92 art. R92/ S - art. R92/ M - art. R92/ L - art. R92/ XL					
Emessa da G.Isetti - Amm.re Unico Pag.: 2/2					
	donning		in compliance		
	dexterit	y	minimum diameter of 5 mm peg		
			collected		

### 5. INSTRUCTIONS FOR USE AND STORAGE

- 1. Please read carefully the information note before using the glove.
- 2. Please keep this information note for the duration of the useful life of the glove.
- 3. The product contains latex rubber. It may cause allergic reactions of type I hypersensitivity and delayed hypersensitivity type IV. In case of reaction, discontinue use and consult a doctor
- 4. Avoid contact with aggressive substances such as fats, oils, acids, hydrocarbons and other chemical agents.
- 5. Before using the product, try it in the real condition of use
- 6. The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed. Wear gloves with dry, clean hands
- 7. Keep the packaging for further information and ensure traceability
- 8. Store the gloves in their original packaging in a dry place
- 9. Do not expose to direct sunlight and heat sources
- 10. The expiry date is indicated on the sales package, and is valid for the product properly stored
- 11. Gloves must be disposed of according to local regulations



