



INFORMATIVE NOTE

Name: REFLEXX T25		Date: 10/05/2021 Rev: 01																			
MANUFACTURER REFLEXX SpA Via Passeri, 2 46019 VIADANA (MN) Tel.: + 39 0375 758891 Fax + 39 0375 464504 info@reflexx.com www.reflexx.com																					
		<table border="1"> <thead> <tr> <th colspan="3">Pack</th> </tr> <tr> <th>Cod./Size</th> <th>Box</th> <th>Carton</th> </tr> </thead> <tbody> <tr> <td>T25/S</td> <td>8032891631676</td> <td>8032891636671</td> </tr> <tr> <td>T25/M</td> <td>8032891631683</td> <td>8032891636688</td> </tr> <tr> <td>T25/L</td> <td>8032891631690</td> <td>8032891636695</td> </tr> <tr> <td>T25/XL</td> <td>8032891631706</td> <td>8032891636701</td> </tr> </tbody> </table>		Pack			Cod./Size	Box	Carton	T25/S	8032891631676	8032891636671	T25/M	8032891631683	8032891636688	T25/L	8032891631690	8032891636695	T25/XL	8032891631706	8032891636701
Pack																					
Cod./Size	Box	Carton																			
T25/S	8032891631676	8032891636671																			
T25/M	8032891631683	8032891636688																			
T25/L	8032891631690	8032891636695																			
T25/XL	8032891631706	8032891636701																			

DESCRIPTION ■ Disposable examination glove, not sterile ■ **Material:** thermoplastic elastomer (TPE) ■ **Shape:** ambidextrous, five fingers ■ **Colour:** blue ■ **Feature:** internally and externally embossed surface ■ **Packing:** 200 pieces/box; 10 boxes/carton ■ **ISO 2859:** AQL 1.5 (level G1) freedom from holes; AQL 2.5 (level G1) major defects; AQL 4 (level G1) minor defects

CONFORMITY AND PERFORMANCES



class I
cat III

Medical Device

Reg EU 2017/745 – EN 455 1,2,3 & 4
T010299 - NON SURGICAL GLOVES- OTHERS

Personal Protective Equipment cat III

0465 ANCI_CIMAC, Via Aguzzafame, 60/b 27029 Vigevano (PV)
Reg EU 2016/425 - EN ISO 374-1:2016 TYPE C - EN ISO 374-5:2016 - EN 374-2:2014 - EN 420:2003 + A1 2009



Reg CE 1935/2004 – Reg EU 10/2011 - DM 21/03/73. To such use carefully read the declaration of suitability for food contact (DICA).

Thickness mm		Palm (+/- 0.01)	0.09
Cod./Size	Length min. (mm)	Width mm (±3)	
T25/S (7)	260	110	
T25/M (8)	260	115	
T25/L (9)	260	120	
T25/XL (11)	260	125	
EN 455-2:2009/A1		Tensile strength	≥3.6 N
EN ISO 374-5:2016		Protection from microorganisms (bacteria and fungi)	Virus protection ISO 16604:2004
		pass	pass

EN ISO 374-1:2016				
Chemical compound:	COD	EN 16523-1:2015 TYPE C		EN 374-4:2013
		Performance level	Breakthrough time	% degradation
Sodium hydroxide 40%	K	6	>480'	17.8%



RISKS FROM WHICH THE DPI IS DESIGNED TO PROTECT:

COD	Protection
K Sodium hydroxide 40% CAS:1310-73-2	Protection from use of inorganic bases

RECOMENDATIONS OF USE Multi-purpose glove to protect the hand and/or the handled product ■ The glove is suitable for contact with pure chemical products for a period of less than the permeation time indicated in the chemical resistance table ■ Results obtained under laboratory conditions may not reflect actual working time **WARNINGS** Before using the product, try it in the real condition of use ■ The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed ■ Wear gloves with dry, clean hands ■ In case of transient skin reactions and hypersensitivity reactions, discontinue use and consult a doctor ■ The user must inform the manufacturer and the relative authority of any incidents occurring during the use of the protective equipment **STORAGE AND DISPOSAL** Store the gloves in their original packaging in a dry place. ■ Do not expose to direct sunlight and heat sources. ■ The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored ■ Gloves must be disposed of according to local regulations ■ Declaration of Conformity on www.reflexx.com